YESOS • STONES • GESSI • PLÂTRES

# DURGUIX

HOJA DE INSTRUCCIONES	ES	3
WORKING INSTRUCTIONS	EN	4
ISTRUZIONI PER L'USO	IT	5
NOTICE D'EMPLOI	FR	6
INFORMACIÓN ADICIONAL ADDITIONAL INFORMATION INFORMAZIONI SUPPLEMENTARI INFORMATIONS COMPLÉMENTAIRES		2 & 7

Consulte nuestra web  
Visit our website

## CARACTERÍSTICAS FÍSICAS • PHYSICAL CHARACTERISTICS CARATTERISTICHE FISICHE • CARACTÉRISTIQUES PHYSIQUES

**100 g / 28 - 30 ml**

Proporción polvo/agua desmineralizada  
Powder/ demineralised water ratio  
Proporzioni polvere/acqua demineralizzata  
Dosage poudre/eau déminéralisée

**20° - 23° C**

Temperatura de trabajo  
Working temperature  
Temperatura di lavorazione  
Température de travail

**60''**

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## DESCRIPTION

Hard natural stone. Type III, high quality ISO 6873.

## USES

Hard natural stone for general laboratory uses, for the fabrication of models in acrylic and Cr-Co technique, opposite models and in the formation of impression trays.

## ADVANTAGES

This product satisfies the most high requirements of the professionals.

- Good expansion
- Hard and even surfaces
- Good definition of details

## INSTRUCTIONS OF USE

Add the powder to the demineralised water following the recommended ratio. The initial consistency of the slurry must be thick. In this moment don't add more water above the recommended.

While the mixing time is complete, the product acquires the suitable fluidity and stability. When this mixing time has finished, the mix must have a soft and high tyxotropic consistency, flowing easily over the model with a low vibration.

After 4 minutes, stop the vibration to guarantee a good crystallisation of the product.

## CAUTIONS

## 1.- USE CAUTIONS

Use always-clean instruments.  
Do not mix the product in a container where investments have been mixed.  
Use always demineralised water to do the mix. The water from the public mains can affect the correct setting process of the gypsum, making it faster or slower.

## 2.- SAFETY CAUTIONS

Do not inhale the powder. It's recommended the use of a protective mask.  
Keep away from children.  
Handle the product in a well-ventilated place.  
The handling of this product has to be done only by qualified people.  
In the event of ingestion, consult a doctor.  
For further information, see the safety data sheet for the product.

## EXPIRY AND STORAGE

## EXPIRY

The stability of the product is 18-36 months depending on the package and respecting the storage conditions. (Note: The expiry date is considered with the product in his original packing and before been opened).

## STORAGE

Keep the packages tightly closed in a dry place away from moisture.  
After each use the package must be closed to prevent the moisture absorption.

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EN

Our recommendations referring to the techniques applied, whether verbally, in writing or in the form of practical instructions, are based on our own experiences and/or tests, and must therefore be considered only as instructions.

For further information, see the safety data sheet of DURGUIX. Order them to your local dealer or visit our website to download them.

IMPORTANT: Check our website for the most updated Instruction Manual version, as this information is subject to continuous review.

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